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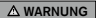
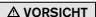
 **WARNING** Warnings regarding possible risks of severe accident or injury.

 **CAUTION** Warnings regarding possible risks of accident or injury.

 **NOTICE** Warnings regarding possible technical damage.

 **INFORMATION** Additional information on the fitting / application.

INFORMATION

Date of the last update: 2014-03-26

- Please read this document carefully.
- Follow the safety instructions.

If you have any other questions, please do not hesitate to contact our team.

1 General Information

1.1 Preface

You have received a custom made silicone prosthesis, which has been adapted to your personal needs. Your prosthetist has informed you about the proper use of your prosthesis.

Please read the following instructions for use carefully. They contain significant additional information regarding the proper use of your prosthesis as well as important safety instructions.

1.2 Areas of Application and Usage Restrictions

CAUTION

Risk of injury as a result of excessive strain. Your prosthesis was designed **solely** for the area of application agreed with you. Sports activities are only permitted after consultation with your physician or with your certified prosthetist.

Your silicone partial foot prosthesis serves as a functional replacement for the amputated foot area and for restoration of the outer appearance. It will help you walk normally and achieve a more natural gait pattern.

Your prosthesis should generally present no discomfort and should be worn during the day. Excessive pressure can lead to complications.

If you have any questions or concerns about your prosthesis, please do not hesitate to contact your certified prosthetist.

1.3 General Safety Instructions

Please observe the following important instructions regarding the use of your prosthesis:

⚠ WARNING

Risk of accident. The ability to operate machinery or motor vehicles with the prosthesis depends on your personal situation. Observing the applicable laws for the operation of motor vehicles is essential. For insurance purposes, you should have your driving abilities examined and approved by an authorised centre. For this purpose please contact the relevant government authority.

⚠ CAUTION

Risk of skin irritations due to non-approved contact substances. To prevent skin irritations, your silicone prosthesis must not come into contact with acids and alkaline solutions, fuels, oils and greases, creams and unguents containing grease, solvents as well as thinners of all kind.

NOTICE

Damage due to excessive strain. Overstressing the product or handling it improperly in any way can damage the material (tearing) and lead to restrictions in functionality. Your silicone prosthesis must be treated with care in order to maintain its outer appearance and functionality.

NOTICE

Do not expose your prosthesis to temperatures above 130 °C. This may result in colour changes. Proper disinfection by boiling remains unaffected (see Section 2.2: "Cleaning and Disinfection").

NOTICE

Damage resulting from using the prosthesis without shoes. Using the prosthesis without shoes could cause surface damage that consequentially leads to malfunctions. In order to avoid damaging the surface, please do not stand on pointed or sharp-edged objects when using the prosthesis without a shoe.

INFORMATION

Please observe the additional information and safety instructions provided by your prosthetist, which will depend on the customised prosthesis used.

1.4 Description of the Construction

INFORMATION

When buying new shoes, be sure to observe the effective heel height predetermined by the prosthesis structure (height difference between the heel and the thickness of the sole in the forefoot area).

The prosthesis consists of a socket to embed your residual limb in and a passively functional replacement compensating for the missing body area. The silicone material used is skin-friendly and physiologically safe.

1.5 Medical and Therapeutic Aspects

CAUTION

Risk of skin damage on the residual limb. Sensitivity problems and skin damage require special attention in regard to the fit of the prosthesis. Examining your residual limb visually and by palpation every day is recommended. Please consult your physician or certified prosthetist in case of any signs of skin damage, even if they are only minor.

As a general rule, physiotherapy is recommended in order to maintain and improve the functionality of the residual limb. Please discuss all measures for maintaining and improving residual limb functionality as well as residual limb treatment and hygiene with your physician or prosthetist.

Depending on your specific indications, complications can occur in individual cases. This may restrict your ability to wear the prosthesis.

1.5.1 Residual limb treatment and hygiene

Individual fabrication and characteristics of your silicone prosthesis ensure a good contact between the prosthesis and body and help prevent skin-damaging friction on the residual limb.

Caring for your residual limb on a daily basis is, nevertheless, essential. Washing it with skin-friendly soap can prevent skin problems.

1.5.2 Fluctuations in volume

As a prosthesis wearer, you should pay special attention to your weight since any fluctuations will have a direct impact on the fit of your prosthesis. Minor volume fluctuations over the course of the day are normal.

If you have additional questions regarding volume fluctuations, please consult your physician or certified prosthetist.

2 Handling, Functionality, Risks

Please verify that your prosthesis is undamaged and functional prior to each use. First-time users in particular have to learn how to handle the prosthesis. Your physician or prosthetist will provide you with more detailed information.

Special attention was paid to the materials used. They were tested for skin tolerance. Nevertheless, should skin changes develop on your residual limb, consult your physician immediately.

2.1 Donning and Doffing

CAUTION

Risk of skin irritation due to air pockets. Air bubbles between the silicone material and skin prevent breathability of the silicone. Evaporated sweat can penetrate the gas permeable silicone material only if the silicone lies directly on the skin. Otherwise, the sweat will condense, accumulate as liquid in the silicone product and can cause skin irritation.

NOTICE

Damage due to finger-nails. Sharp-edged finger nails can damage the thin brim of the prosthesis when donning and doffing it.

Silicone partial foot prostheses are worn directly on the residual limb. Your prosthetist has instructed you in the technique of donning. The following donning technique has proved ideal for silicone partial foot prostheses:

- Because of the high surface friction of the silicone material, it may be necessary to make the residual limb more slippery with a neutral, water-based and perfume-free skin lotion. Apply an additional bit of Procomfort Gel to the inner brim of the prosthesis.
- Slide your residual limb into the access opening with only light pressure.
- Slowly continue to slide your residual limb into the silicone partial foot prosthesis.
- Verify the proper fit.
- Press or massage out any air bubbles between the silicone partial foot prosthesis and residual limb.

To remove the silicone partial foot prosthesis, proceed in the reverse sequence. Slide your finger between the silicone material and skin to allow air flow in. It will then be easy to pull off the prosthesis.

2.2 Cleaning and Disinfecting

INFORMATION

Please observe the important instructions in Section 1.3 "General Safety Instructions" regarding forbidden contact substances.

NOTICE

Damage caused by improper cleaning agents. To prevent material damage, do not use **any** solvents such as thinner, acetone, benzene, etc.

NOTICE

Damage due to inadequate cleaning. Daily cleaning with a neutral soap will remove sweat components such as skin grease, amino acids and sugar from your silicone prosthesis. Without such daily cleaning, your silicone prosthesis might suffer long-term damage.

Your prosthesis requires regular care. For this purpose, daily cleaning with water and neutral soap is necessary to reliably remove any sweat residue.

It is important to thoroughly dry all cleaned surfaces and to make sure that no soap residue remains on the product since this could cause skin irritation on your residual limb.

Substances that have penetrated the surface must be removed by boiling the product once a week. For this purpose the silicone partial foot prosthesis is boiled out for 30 minutes. Following this treatment it will be sterile once more.

If you have any questions on skin diseases and disinfection, please consult your physician or prosthetist.

2.3 Inspection

INFORMATION

Liability is assumed for the product only in the form Ottobock delivered it. The liability only applies to services provided by Ottobock.

Please pay attention to changes that appear in the material of your prosthesis – especially at the thin brims. If such changes occur, please contact your certified prosthetist immediately.

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Otto Bock HealthCare GmbH